




METRASTOP

DUAL LEIGHTWEIGHT BALLOON

(weight 70gr)

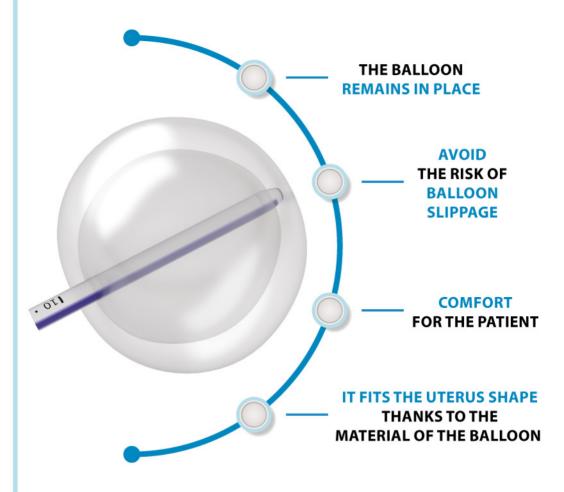

Postpartum Balloon

STIFF CATHETER

EASE OF INSERTION


(without using a forceps)

EASE OF POSITION

EASY AND QUICK CATHETER REPOSITIONING IN CASE OF MISPLACEMENT

VERY FAST PROCEDURE OF INFLATION AND DEFLATION

The inflation valve of the transparent lumen enables to maintain the syringe connected during the inflation process

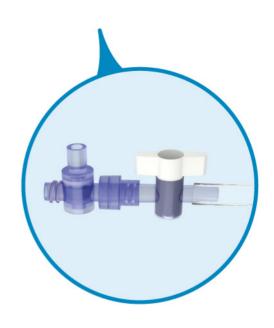


HOW DOES IT WORK?

1) BALLONS INFLATION

1° EXTERNAL BALLOON

60 ML syringe of sterile saline solution → BLUE LUMEN: "WATER" (close the stopcock)

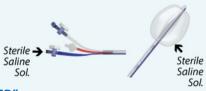

2° INTERNAL BALLOON

Same 60 ML syringe of air → TRANSPARENT LUMEN: "AIR INFLATION" MAX 420 CC (CA 7 PUMPS CONTINUOUSLY) (close the stopcock)

2) CONNECT THE DRAINAGE BAG



Fluid collection bag (included) → RED LUMEN (DRAINAGE)

3) CATHETER REMOVAL



1° INTERNAL BALLOON

TRANSPARENT LUMEN -> Remove the valve and leak the air out

2° EXTERNAL BALLOON

BLUE LUMEN → Aspirate the content with a syringe

- It is also possibile to cut the catheter lumens in order to deflate the balloons more quickly -

COMPANY PROFILE

RI.MOS. began designing and selling gynecology products in 1988.

Since then, it is the only Italian manufacturer of a complete range of disposable products for gynecology / obstetrics and IVF.

R&D and engineering of all products are carried out internally and the manufacturing process involves highly experienced operators and suppliers in the manufacturing of medical devices.

Thanks to a modern production technology, **RI.MOS.** products are assembled by trained operators in **ISO 7** and **ISO 8** clean rooms.

RI.MOS quality system is certified according to ISO 13485:2003.

RI.MOS. is able to realize small customized productions in order to meet every need of the private gynecologist and Hospital.

RI.MOS. is the only Italian company actively engaged in the development and production of a complete range of disposable devices used for artificial insemination and IVF.



Order Info



External balloon to be inflated with 60 ml syringe of **sterile saline solution** (syringe included)



Internal balloon to be inflated with max 420 cc of **air** (ca 7 pumps)

Code	Description	Lenght
720264	Box of 2 pcs. Supplied in single pouch, single use* ETO sterilized and C C marked.	54 cm
*A 60 ml syringe and a drainage bag for the hemostasis monitoring included		



Refer to the Instructions for Use for complete information on product usage and a complete list of precautions, warnings and contraindications. www.meddiscount.at

